## AGA/10 - Air Gel Ankle

EN ANKLE/FOOT ORTHOSIS

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ES ORTESIS DE TOBILLO/PIE

IT ORTESI DI CAVIGLIA/PIEDE

NL ENKEL-/VOETORTHESE

DK ANKEL-/FODORTOSE

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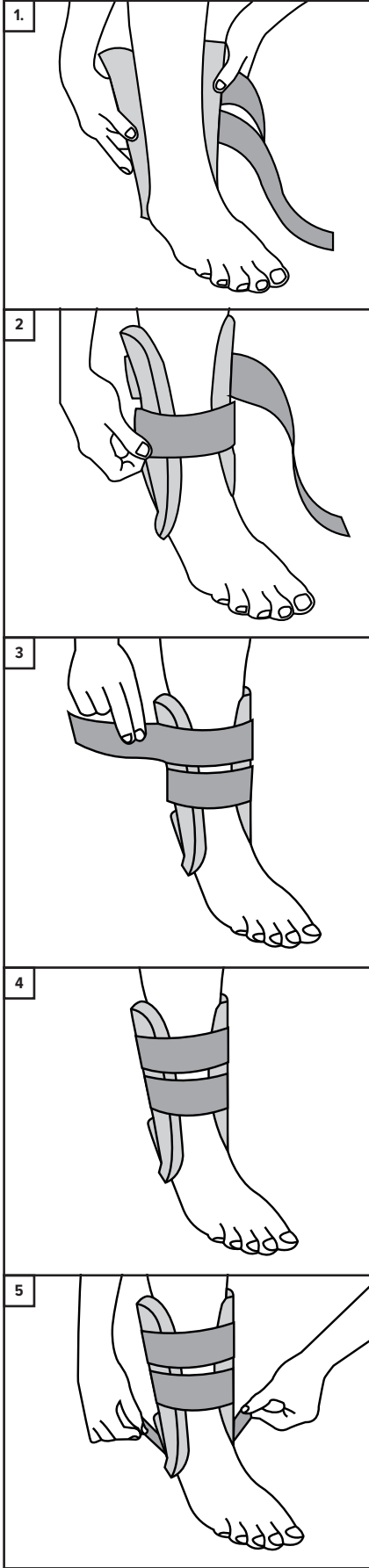
SE ORTOS, FOT/FOTLED

NO ORTOSE, ANKEL/FOT

PL ORTEZA GOLEŃ/STOPA

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Universal ankle stirrup.

#### Usage Information (See diagram)

1. Wrap brace around ankle, tucking brace under the heel as shown.
- 2 - 4. Apply straps to secure brace, ensuring a firm, comfortable fit without feeling tight.
5. Use the side heel straps to tighten brace around heel for a more secure fit. Re-adjust as required, ensuring a firm, comfortable fit without feeling tight.

#### Indications for Use

- Mild to moderate ankle sprains.
- Chronic ankle instability.
- Post cast and post walker support.
- Ankle injury rehabilitation.
- Post-surgical.

#### Contraindications

- Do not apply the product in direct contact with broken skin.

#### Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

#### Washing and Care Instructions

1. Hand wash liner with mild detergent.
2. Wipe down the shell with clean damp cloth and mild detergent and allow to air dry.
3. Check for any degradation that may affect use.
4. Periodically check Hook/Loop fasteners are operating securely.

#### Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

#### Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

#### Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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Promedics Orthopaedics Ltd.