## LCRA - Lacer Ankle

### EN ANKLE/FOOT ORTHOSIS

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### ES ORTESIS DE TOBILLO/PIE

### IT ORTESI DI CAVIGLIA/PIEDE

### NL ENKEL-/VOETORTHESE

### DK ANKEL-/FODORTOSE

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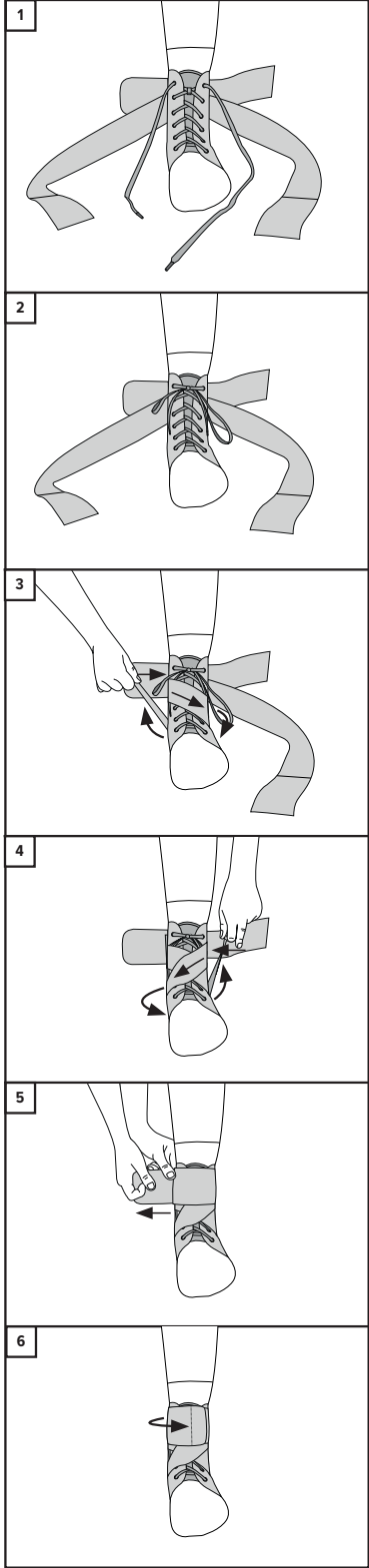
### SE ORTOS, FOT/FOTLED

### NO ORTOSE, ANKEL/FOT

### PL ORTEZA GOLEŃ/STOPA

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#### ENGLISH

##### Product Description

A low-profile lacer ankle brace providing ankle stabilisation with figure of 8 straps and elastic ankle cuff.

##### Usage Information (See diagram)

- The ankle brace is designed to be worn over a sock. Position heel in the hole at the rear and bottom of the brace. Lace the ankle brace and tie snugly.
- Wrap the inside (medial) strap across the top of the foot. Continue by wrapping the strap under the foot and heel. Secure the hook to the loop on the inside of the ankle. Pull strap tight to ensure proper performance.
- Wrap the outside (lateral) strap across the top of the foot. Continue by wrapping the strap under the foot and heel. Secure the hook to the loop on the outside of the ankle.
- Finish by wrapping the hook / loop strap around the lower leg at the top of the brace.
- Ankle brace fitted.

##### Indications for Use

- Mild to moderate ankle sprains.
- Ankle injury rehabilitation.

##### Contraindications

- Do not use over open wounds.
- Any known allergy and/or hypersensitivity to any material listed in product composition.
- Caution should be used if prescribed to patients with diabetes, vascular deficiency and neuropathy.

##### Warnings and Precautions

We recommend the initial fitting of this device be conducted by a suitably qualified healthcare professional who will advise the period of use.

- Carefully read all instructions and warnings prior to use.
- Follow all instructions to ensure proper performance of the device.
- Do not use if liniments, ointments, gels, creams or any other substances have been applied to the affected area.
- Do not re-use for another patient, doing so risks cross infection and can compromise product integrity.
- Perform regular skin and circulation checks, especially patients with diabetes, vascular deficiencies and neurological conditions.
- Should any adverse reactions occur, please cease use and contact your healthcare professional or provider of this product.
- The durability of the device may be compromised by certain factors, e.g., objects with sharp edges or damage to the hook and loop fasteners. To prevent this the hook and loop fasteners should always be fastened when the device is not being worn or when being washed.

##### Washing and Care Instructions

- Hand wash brace in cool water using mild detergent.
- Rinse and dry flat.
- Do not spin or tumble dry.
- Do not use fabric conditioner at any time.
- Check for any degradation that may affect use.

##### Product Family Composition

Polyamide, Polyurethane, Polyester, Copper, Polyethylene, SBR.

##### Storage and Transport Conditions

Store in a cool, dry place out of direct sunlight, in the original packaging.

##### Recycling and Disposal

Packaging and constituent parts should be recycled or safely disposed in accordance with local or national laws.

##### Serious Incident

Report any serious incident to the manufacturer and the competent authority of the EU Member state, or country in which you reside.

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©2023 Promedics Orthopaedics Ltd. All Rights Reserved.
Promedics Orthopaedics Ltd.